

STEALTHSTATION™ CRANIAL AND SYNERGY CRANIAL™ DEPTH GAUGE INACCURACY PREVENTION STEPS

Medtronic

STEALTHSTATION™ CRANIAL AND SYNERGY CRANIAL™

DEPTH GAUGE INACCURACY PREVENTION STEPS

An anomaly currently exists in your S7 Cranial software in the biopsy procedure type. Until a software update is made available, please ensure you are following the steps below to prevent the issue from occurring:

Per the Cranial Software Pocket Guide, **ALWAYS USE the Biopsy Needle Mechanical Depth Stop.**



Do not set a projection longer than the length of the surgical plan.

Ensure **Navigate Instrument Tip** setting is selected prior to locking the trajectory and subsequently navigating the instrument (see image to the left).

If you have questions related to this issue, please contact Technical Services for help at: 1-800-595-9709
RS.NAVTECHSUPPORT
@MEDTRONIC.COM