Março, 2026

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© 2026 QuidelOrtho Corporation. All rights reserved. All trademarks are the property of QuidelOrtho Corporation or its subsidiaries.

*Quidel Corporation (Quidel) and Ortho Clinical Diagnostics (Ortho), wholly owned subsidiaries of QuidelOrtho Corporation, are transitioning to our new logo and brand. Due to legal and regulatory requirements for diagnostic products, you may continue to see the names and brands of Quidel and Ortho in addition to QuidelOrtho on our packaging, contracts, and marketing materials.*

## Appendix 1: Transition to VITROS Chemistry Products Calibrator Kit 32

For more information on transitioning to VITROS Chemistry Products Calibrator Kit 32, please refer to the product Instructions For Use.

To reload the current Assay Data Disk (ADD):

For e-Connectivity customers *only*: The ADD stays in the inbox of the VITROS System even

if the  icon is not displayed on the System Status screen, until a new ADD becomes available through e-Connectivity. During the ADD load process, select the 'Downloaded files' option at the 'Load System Information' dialog and then select 'Next'. Confirm the ADD to be loaded is the intended DRV. Do not load a DRV that is lower than what is currently loaded on your VITROS System.

Refer to the VITROS System user instructions as follows:

1. On the Load Method dialog, select the 'All Assay Data' option.
2. When prompted, select 'Change Kit' and then touch 'Next' and wait for the data to load. This may take a few minutes.
3. When prompted, select '32' to configure the VITROS System for VITROS Chemistry Products Calibrator Kit 32, then touch 'Next' and wait for the data to load.